




VersaMed iVent 201 On/Off Switch Replacement

How to replace the power switch on a VersaMed iVent 201 ventilator

Written By: Miriam Miller

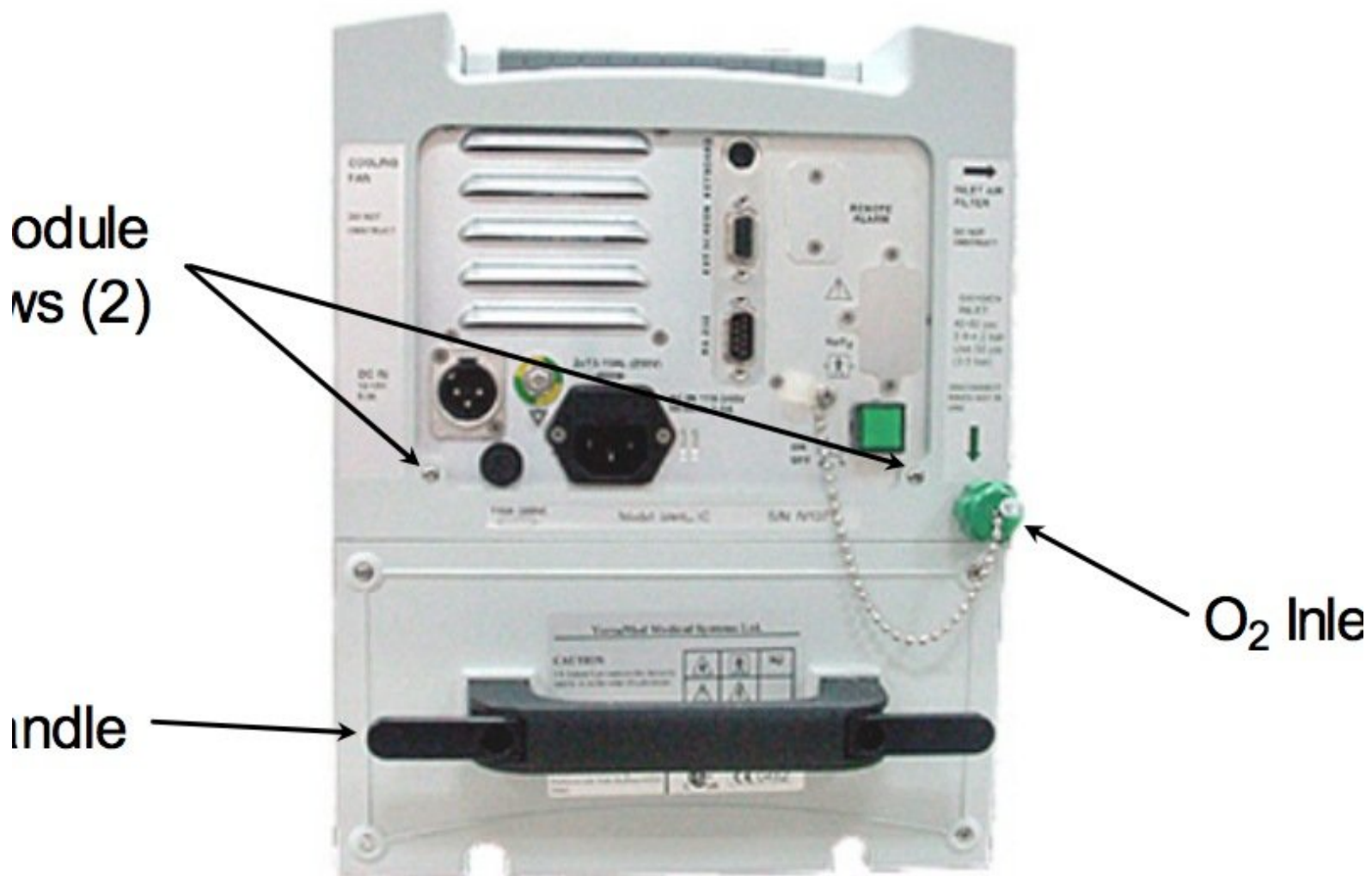

INTRODUCTION

This guide provides detailed steps on fixing a mechanical issue within your ventilator on/off switch. This procedure will be useful if your unit will only power up as long as you hold the power switch in. This is a problem, as the device is designed to power up on a single push of the switch. To perform the following procedure, you will need a pair of pliers or tweezers. This is an easy fix and may take around 5 minutes.


For more information on replacing the switch, please refer to **page 133** in the service manual.



TOOLS:


- [Tweezers](#) (1)
-

Step 1 — On/Off Switch

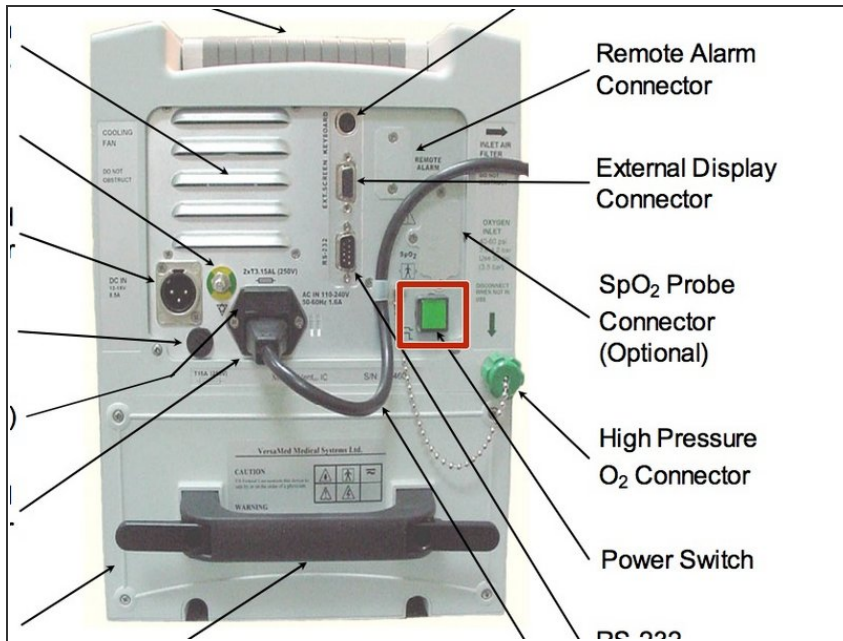


- Use a pair of slim pliers or tweezers to grip the cap of the power switch button. Pull it straight out.

Step 2

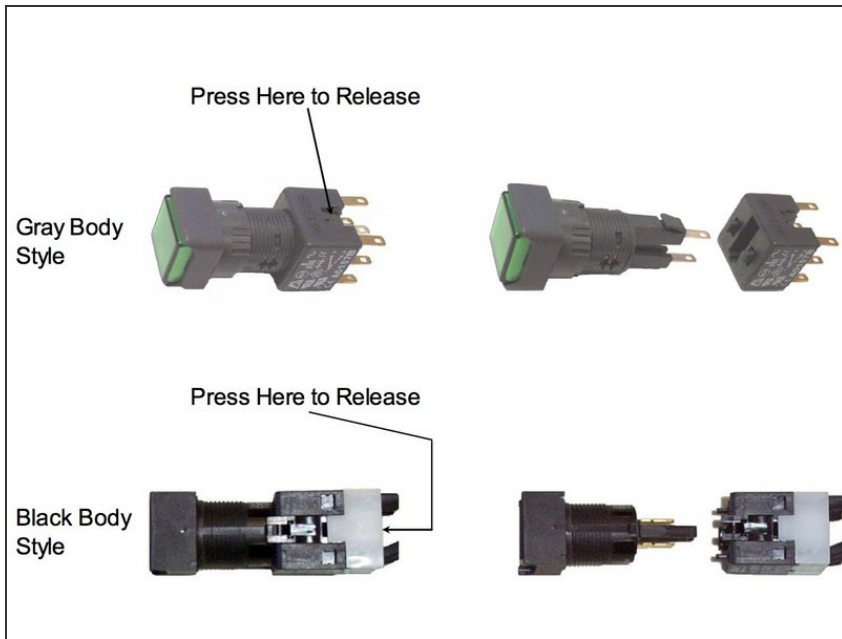

- Examine the exposed area and look for a gray, metallic ring. If it is present, use the pliers or tweezers to remove it. If no ring is found, skip this step.

Step 3

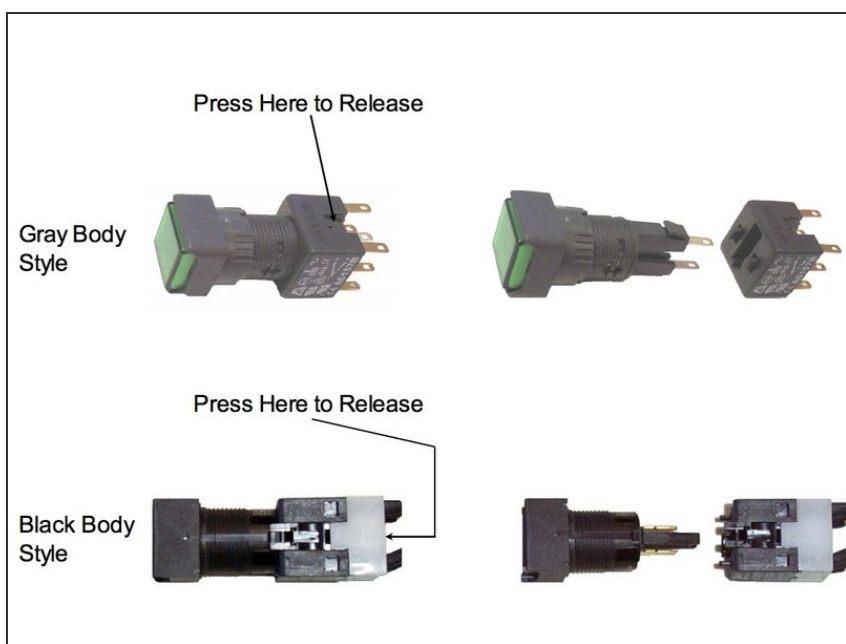

- ★ For reinstallation, locate the white dot on the exposed surface, and line this dot up with the slot in the middle of the button. Press the button straight in.

To reassemble your device, follow these instructions in reverse order.